



LEAVING YOUR PRACTICE FOR A LITTLE WHILE,
MAKE SURE FOLLOW THESE SHUTDOWN
PROCEDURES

Description

[vc_row][vc_column][vc_column_text]

Chair Shutdown procedure for Stern Weber and Castellini

- Run the unit through a cleaning cycle Autosteril / Bioster or 2 min Flush cycle depending on the version you operate on your unit.
- Remove Water bottle and Drain lines of all water by activating



dynamic instrumentation

- Drain Air condensate located either at the base of the Chair or under the spittoon
- Run Suction Cleaning Procedure
- Clean all filters and gently apply O ring grease or Vaseline to all rubber seals
- **DO NOT LEAVE ANY SUCTION, SPITTOON FILTERS, SUCTION ENDS, IN ANY CLEANING SOLUTION, OF ANY KIND, OTHER THAN WATER**
- Drain your spittoon valve by depressing the Drain activation switch inside the spittoon
- Switch Off the unit on the Chair and on the wall

Procedure for Suction Motor shut down Durr and Cattani

- Run your cleaning product through the Suction lines
- Run 2 x volume of Clean Water afterwards
- Run the Suction motor for 10 minutes with only air running through the suction tubes
- Switch of the Suction Motor and remove the amalgam pot and clean and drain
- Clean suction filter
- Refit amalgam pot and filter and gently apply O ring grease or Vaseline to all rubber seals
- Switch Off the Suction unit



Compressor Shutdown procedure

- Turn Compressor Off
- Drain the Tank of all Air with Tank Drain Valve
- Drain the Dryer Column where applicable

[/vc_column_text][[/vc_column]][[/vc_row]

Category

1. Dental